CATALOGUE

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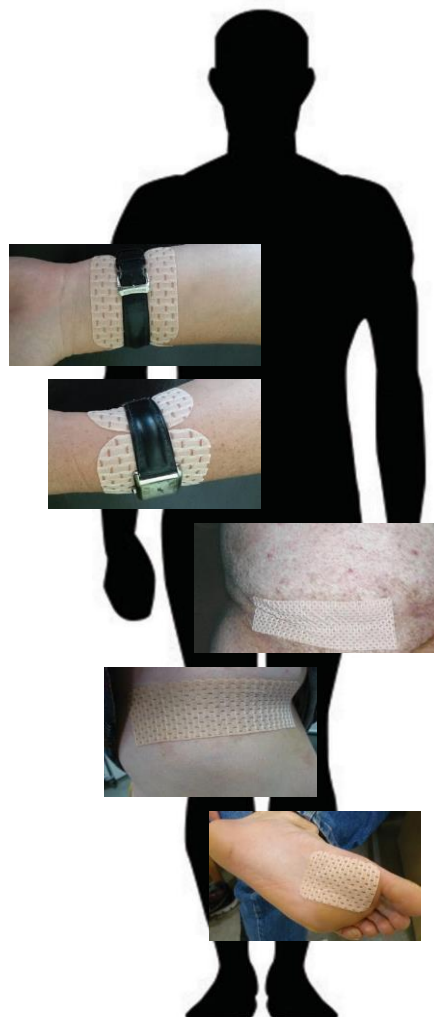
INTERNATIONAL CONSENSUS

Best Practice Guidelines

Skin and wound care in
EPIDERMOLYSIS BULLOSA

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What is the dressing?

Spycra Protect combines a gentle soft silicone adhesive with a bi-elastic top layer.

The two way stretch of the dressing allows friction to be taken up by movement of the dressing thus saving the skin from shearing forces.

Do I need a dressing underneath Spycra Protect?

No, this dressing can be put directly onto intact skin, minor wound or blister site.

Will I need a dressing over the top?

Not if you are using this for protection of fragile skin or on a minor wound or blister. If the wound has a lot of fluid then Spycra Protect may not be suitable.

Will the dressing tear my skin when I take it off?

The silicone adhesive is very gentle, but if you feel it is pulling at all or feels stuck then use a silicone medical adhesive remover (such as Appeal, Peelasey or Nilact).

Can I cut the dressing?

Yes, cut it with sharp scissors before removing the backing paper.

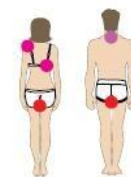
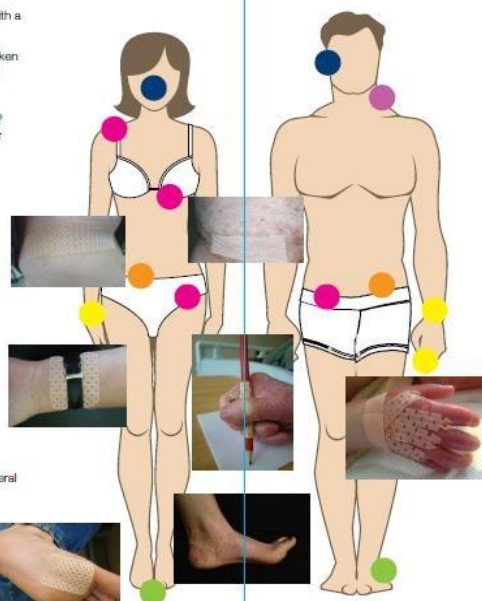
How long can the dressing stay in place?

This depends on how often you like to change your dressings and how wet or dirty the wound is if placed on a wound.

It is fine to change it daily, but it can be left in place for several days to allow healing to take place in the case of wounds.

I am allergic to silicone – can I use Spycra Protect?

No, this product uses a silicone adhesive so if you are one of the unlucky few with silicone sensitivity then do not use this dressing.

Ideas for using Spycra Protect:

- On minor blister sites
- On healed areas of skin for protection
- On the bridge of your nose or around your ears to prevent glasses from rubbing
- Under tight underwear such as bras (under straps and below breasts)
- Bra Straps
- Waist bands/underwear
- Under watch straps
- Between fingers in dystrophic EB to delay webbing
- Around fingers when holding pencils
- Under rings
- On bottoms/Buttocks
- Under skin folds
- On sites of friction such as the feet in EB simplex
- Around heels / base of feet
- Spycra Protect can also be put onto equipment such as glasses or pencils
- Under and around collars

Letteratura Epidermolisi Bollosa

Great Ormond Street **NHS**
Hospital for Children
NHS Trust

The Use of a Novel Bi-Stretch Silicone Dressing in the Management of Children with Epidermolysis Bullosa

Jackie Denyer Lead EB Clinical Nurse Specialist

Epidermolysis Bullosa (EB) is the term used for a large group of genetically determined blistering skin disorders. There are 4 major types of EB with many subgroups. The common factor is a marked skin fragility with minimal friction and trauma resulting in blistering or skin loss. Fragility of the mucous membranes together with multiple co morbidities lead to progressive disability from scarring or reduced life expectancies in severe forms.

The aim of the study was to evaluate the effectiveness of the dressing's ability to reduce friction and trauma. The dressing (Spycra Protect) is a bi-elastic dressing with the elastic being active in both width and stretch. This bi-elasticity diminishes the effects of friction. This characteristic together with the gentle silicone adhesion suggested it would be effective and well tolerated by children with Epidermolysis Bullosa.

The dressing is non-absorbent, making it suitable for protection from trauma, blister sites and minor wounds only. A top layer of lycra fibres gives the dressing a smooth surface which minimises trauma from the child rubbing with the dressing to relieve itch.

A total of 45 children representing all forms of EB and with the age range of 4 months to 15 years were selected for the study.

The dressing was applied over intact skin to offer protection from mechanical trauma or over blister sites.

In all cases the dressing proved easy to apply and remove and was comfortable during wear time. No trauma resulted from the removal of the dressing and adhesive removers were not required.

Results

The dressing was helpful	Pain relief from blister sites	Reduction in friction	Effective protection from trauma	Effective retention of finger dressings	Healing of minor erosions
38 children (EB simplex)	31 children (EB simplex)	31 children (EB simplex)	9 children (EB simplex and dystrophic EB)	2 infants (Junctional EB)	1 infant (Kindler syndrome)

5 children who have EB simplex and suffer from hyperhidrosis found the dressing difficult to keep in place, which is a problem with the majority of dressings in this group of children.

Conclusion

Reduction of trauma is paramount in all forms of EB, both severe and mild. Using a dressing with bi-stretch properties reduces friction and shear. The protective silicone dressing has proved effective for this challenging patient group.

This Poster was supported by Bullen Healthcare, the distributor of Spycra Protect.

Case study 1

Freddy is a 5 year old with severe generalised recessive dystrophic EB.

He is very restless at night and rubs his face constantly creating blisters and areas of skin loss. His parents now use bi-stretch silicone dressings to protect his face reducing the amount of damage caused.

Case study 2

Aisha is a 4 year old girl with severe generalised EB simplex.

She has extensive blistering on the soles of her feet which are painful and restrict her mobility. Applying a sheet of bi-stretch silicone dressing reduces the friction and pain and her mobility has improved.

Case study 3

Amy is a 3 year old girl with severe generalised dystrophic EB.

She uses bi-stretch silicone dressing applied to her finger web spaces to help delay digital fusion. The dressing is light and therefore does not restrict function. It can be applied over minor wounds.

Letteratura Epidermolisi Bollosa

Guy's and St Thomas' **NHS**
NHS Foundation Trust

debra
for people whose
skin doesn't work
we do

Use of Spycra Protect in Epidermolysis Bullosa (EB)

Jane Clapham Lead EB CNS • Chris Bloor EB CNS • Guy's & St Thomas' NHS Trust

Introduction

EB is a group of inherited fragile skin disorders for which there is no known cure. There are 4 main types ranging in severity however the commonalities are characterised by blister formation in response to the slightest friction or trauma. Treatment aimed at minimising pain & reducing risk of infection. Open lesions occur at the site of blistering which may result in secondary infections; blisters are not self-limiting & must be lanced.

EB Simplex (EBS) is often perceived as a milder form of EB however it can be very disabling, affecting the quality of life. Blistering is primarily to hands & feet, but can occur on any part of the body exposed to friction. A major problem for those affected is protection of vulnerable areas such as hands, feet & bony prominences. Good protection can reduce the number of blisters & severity of skin damage. Removal of many adhesive medical products can cause further skin damage. Soft silicone products are mostly well tolerated as can usually be removed without further pain & damage to fragile skin.

Method

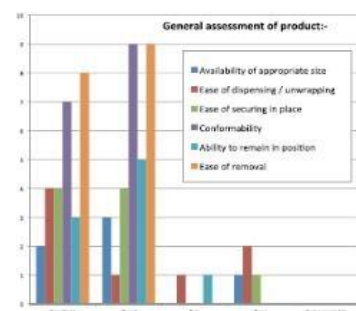
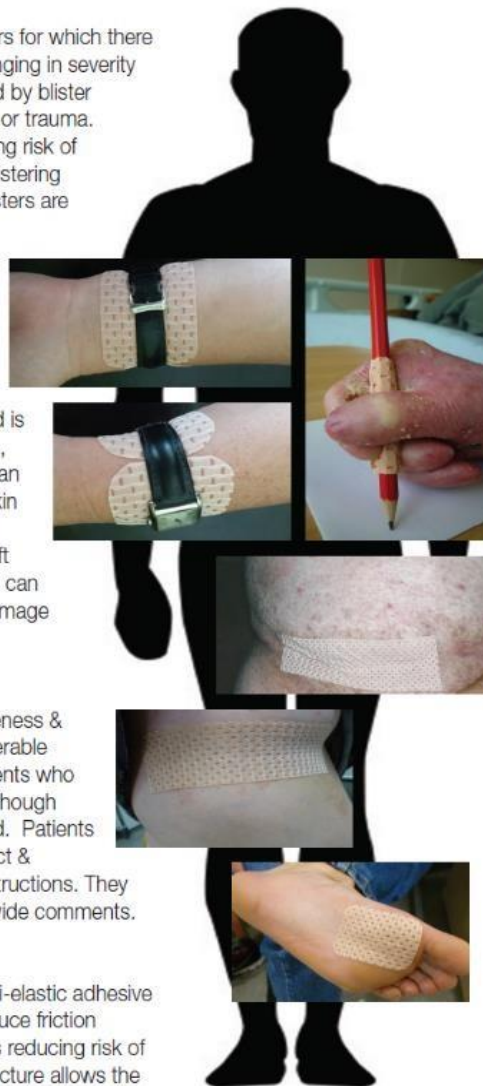
Aim of this evaluation was to assess effectiveness & tolerability of Spycra Protect in this very vulnerable group, whilst maintaining comfort. Most patients who evaluated Spycra Protect suffer with EBS, although patients with other types of EB were included. Patients were provided with a supply of Spycra Protect & instructions on use as per manufacturers instructions. They were asked to complete an evaluation & provide comments.

The product

Spycra Protect is a soft, one-sided silicone bi-elastic adhesive dressing. The bi-elasticity is purported to reduce friction through complex 3D weave of the fabric thus reducing risk of blistering from shearing forces. The pore structure allows the dressing to breathe; the non adherent dressing can be easily removed or repositioned without pain or further skin damage. It can be used directly onto intact skin, minor wounds or blister sites. It is not designed to absorb, if the wound is wet an alternative dressing would be required.

Discussion

A big advantage of Spycra Protect is its conformability; the thinness allows the dressing to be used on areas such as the feet. Feedback suggested the dressing could be a stickier. However there is a fine balance between a dressing not sticking & falling off or being too sticky it causes further skin damage.

Results

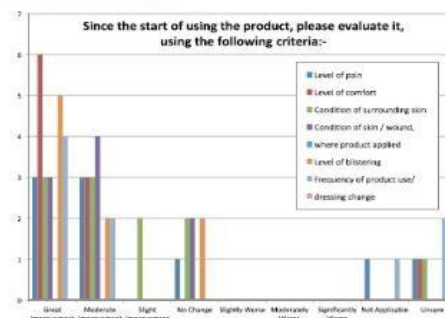
Spycra Protect was used on the following areas

- Feet, toes, heels, ankles
- Bra & waste band areas
- Elbows
- Sacrum

Most patients changed the dressing daily although some re-used if it had been placed on intact skin. Innovative & interesting uses were identified such as securing dressing post operatively in difficult areas.

Conclusion

Results demonstrate the dressing is well accepted & comfortable. Most patients found it excellent, protecting their fragile skin, some found it eliminated skin damage from friction altogether. Evidence also suggests it is best suited to patients with EBS, those with other types of EB did not find it as useful. This dressing is a valuable addition to people with EB.

This Poster was supported by Bullen Healthcare, the distributor of Spycra Protect.

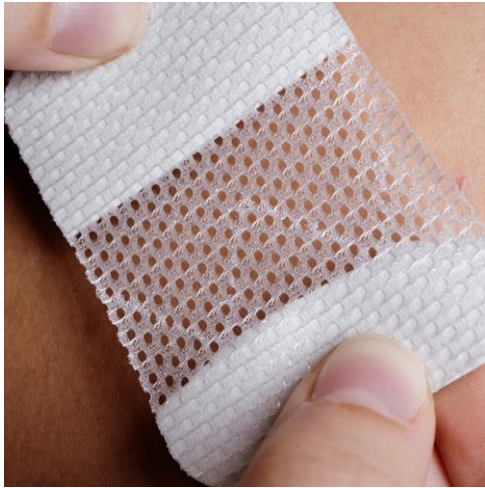
BULLEN
Generations of Quality Healthcare

SPYCRA[®]

Contact

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Product	Reference	Size (cm)	Size (in.)
	WCL - 575	5 cm x 7,5 cm	2.0 in. x 3.0 in.
	WCL - 7510	7,5 cm x 10 cm	3.0 in. x 4.0 in.
	WCL - 1010	10 cm x 10 cm	4.0 in. x 4.0 in.
	WCL - 1018	10 cm x 18 cm	4.0 in. x 7.0 in.
	WCL - 1830	18 cm x 30 cm	7.0 in. x 12 in.

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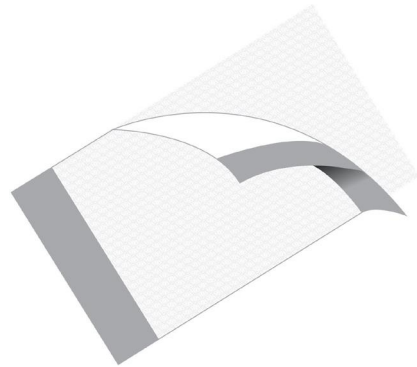
Film

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New design

Product	Reference	Size (cm)	Size (in.)
	SF - 68	6 cm x 8 cm	2.4 in. x 3.1 in.
	SF - 1012	10 cm x 12 cm	3.9 in. x 4.7 in.
	SF - 1020	10 cm x 20 cm	3.9 in. x 7.9 in.
	SF - 1520	15 cm x 20 cm	5.9 in. x 7.9 in.

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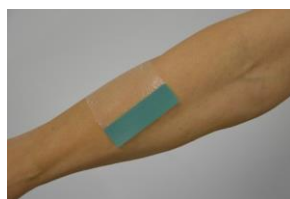
SPYCRA[®]

Roll

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Product	Reference	Size (cm)	Size (in.)
	SR - 55	5 cm x 5 m	2.0 in. x 197 in.
	SR - 105	10 cm x 5 m	4.0 in. x 197 in.
	SR - 155	15 cm x 5 m	6.0 in. x 197 in.

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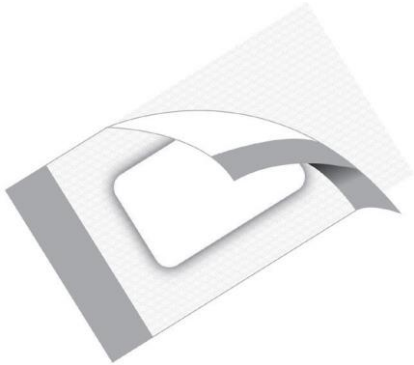
SPYCRA®

Post-op Border

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Product	Reference	Medicazione		Tampone	
		Size (cm)	Size (in.)	Size (cm)	Size (in.)
	SPO - 6080	6 cm x 8 cm	2.4 in. x 3.1 in.	3 cm x 4 cm	1.2 in. x 1.7 in.
	SPO - 60100	6 cm x 10 cm	2.4 in. x 3.9 in.	3 cm x 6 cm	1.2 in. x 2.4 in.
	SPO - 80100	8 cm x 10 cm	3.1 in. x 3.9 in.	4.5 cm x 6 cm	1.8 in. x 2.4 in.
	SPO - 90150	9 cm x 15 cm	3.5 in. x 5.9 in.	4.5 cm x 10 cm	1.8 in. x 3.9 in.
	SPO - 90200	9 cm x 20 cm	3.5 in. x 7.8 in.	4.5 cm x 14 cm	1.8 in. x 5.5 in.
	SPO - 90250	9 cm x 25 cm	3.5 in. x 9.8 in.	4.5 cm x 18 cm	1.8 in. x 7.0 in.
	SPO - 100300	10 cm x 30 cm	3.9 in. x 11.8 in.	5 cm x 22 cm	1.9 in. x 8.6 in.
	SPO - 100350	10 cm x 35 cm	3.9 in. x 13.8 in.	5 cm x 27 cm	1.9 in. x 10.6 in.

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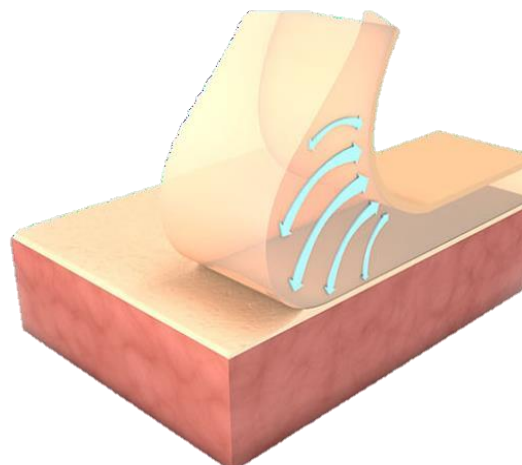
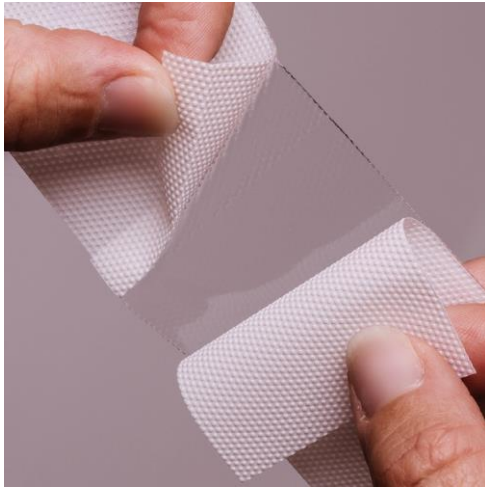
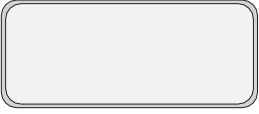
SPYCRA®

Scar

Product	Reference	Size (cm)	Size (in.)
	SPY SCAR - 3070	7 cm x 3 cm	2.8 in. x 1.2 in
	SPY SCAR - 12030	12 cm x 3 cm	4.7 in. x 1.2 in.

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